



PRODUCT SHEET

CorTTape®

FIXED

NON-ADJUSTABLE LOOP CORTICAL
FIXATION SYSTEM FOR ACL RECONSTRUCTION



Product delivered in double packaging
and sterilized by Oxide Ethylene.
Do not re-sterilize.

Product description

The device is composed of:

- Two non-absorbable polyester pulling threads (1 blue/white and 1 white),
- A continuous white braided Ultra High Molecular Weight Polyethylene loop (UHMWPE),
- A titanium plate (length: 12 mm, width: 4 mm, thickness: 1.5 mm, Ti-6Al-4V ELI).

Product Features

Several loop lengths are available:
15 mm, 20 mm, 25 mm, 30 mm and 35 mm.

Indications

CorTTape® FIXED is indicated for fixation of bone or of soft tissue in orthopedic procedures requiring reconstruction of a ligament or tendon.

CorTTape®

FIXED

NON-ADJUSTABLE LOOP CORTICAL
FIXATION SYSTEM FOR ACL RECONSTRUCTION




Choice of loop length of CorTTape® FIXED

Calculation to determine the length of the loop:

Total length of the femoral tunnel

- ⊖ length of the bone cavity
- + 12 mm* (always round up to the larger size).

* the additional 12 mm is necessary for the tilting of the plate.

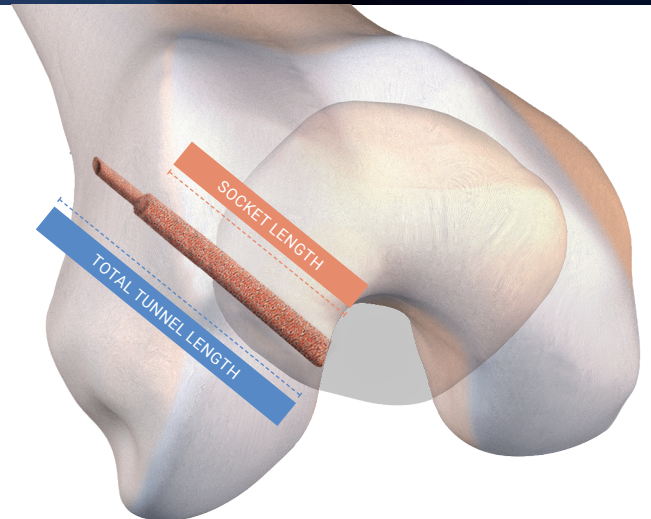

Example:

Total tunnel length 42 mm

- ⊖ Length of the femoral compartment 20 mm
- + 12 mm rocker
- = 34.

Choose a 35 mm Corrtape® FIXED.

Mechanical test of the device alone

Tensile strength: 1500 N

FH ref.	Supplier ref.	Designation
271 891	OBCL-15-Ti	CorTTape® FIXED 15 mm
271 892	OBCL-20-Ti	CorTTape® FIXED 20 mm
271 893	OBCL-25-Ti	CorTTape® FIXED 25 mm
271 894	OBCL-30-Ti	CorTTape® FIXED 30 mm
271 895	OBCL-35-Ti	CorTTape® FIXED 35 mm



DISTRIBUTEURS
DISTRIBUTORS

GROUPE
FH ORTHO™

FR, FH ORTHO SAS
3 rue de la Forêt - Zone Industrielle
BP 50009
68990 Heimsbrunn CEDEX - FRANCE
Tél. +33 (0)3 89 81 90 92
Fax : +33 (0)3 89 81 80 11
info@fhortho.com
www.fhortho.com

USA, FH ORTHOPEDICS INC.
OrthoEx
7327 E Tierra Buena Lane
Scottsdale, Arizona 85260 - USA
Phone: +1 (412) 965-0950
customerservice@fhortho-us.com
www.fhortho.com

PL, FH ORTHO POLSKA
Ul. Garbary 95/A6,
61-757 Poznan - POLSKA
Phone: +48 61 863 81 27
Fax: +48 61 863 81 28
biuro@implants24.pl
www.fhortho.com