



# Knee+ 1.6


## Surgical technique

---

A new vision for computer assisted orthopedic surgery.



Only **3 steps** before navigation:

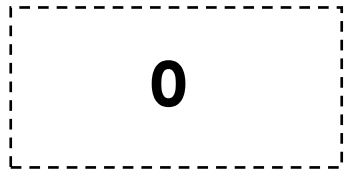

HMD\* setup

**Camera lenses cleaning**  
Accelerometer calibration  
**Wifi & protocol Config.**  
**Protocol customization**




Operate side selection

Slim Bone reference installation

Landmarks acquisition

**Cutting guide Navigation**

Surgical report sending

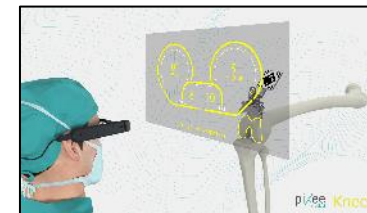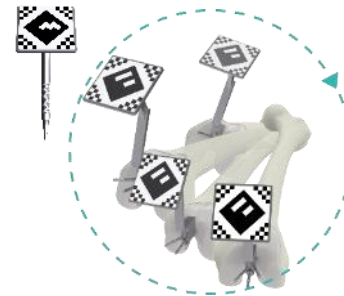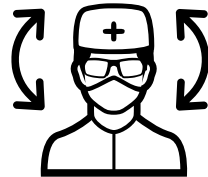



\*Head Mounted Device setup



The VUZIX M400 camera films during this step



The external camera will take over the VUZIX M400 from this step

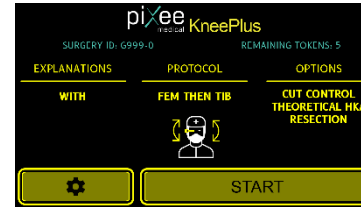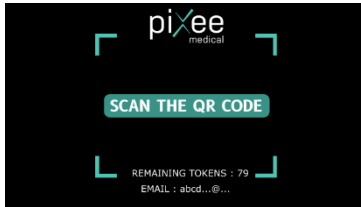

Accelerometer  
Calibration


Launch of  
KneePlus


Serial number  
check

Protocol  
configuration

Leg selection

First  
assembly

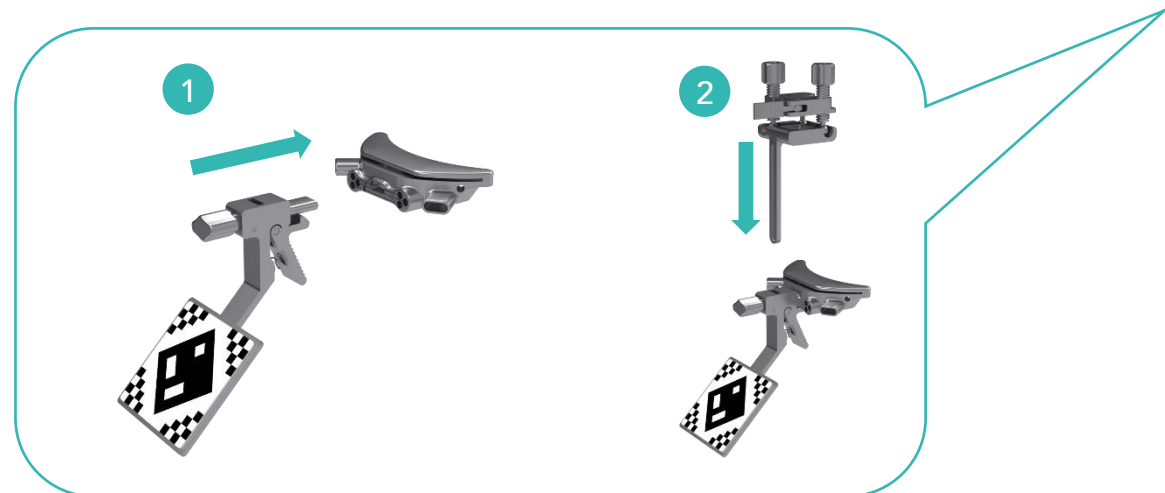





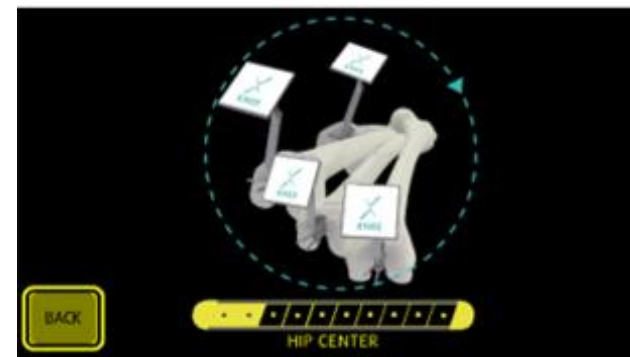


## Modular Pointer installation

## Slim Bone reference installation

## Hip center registration

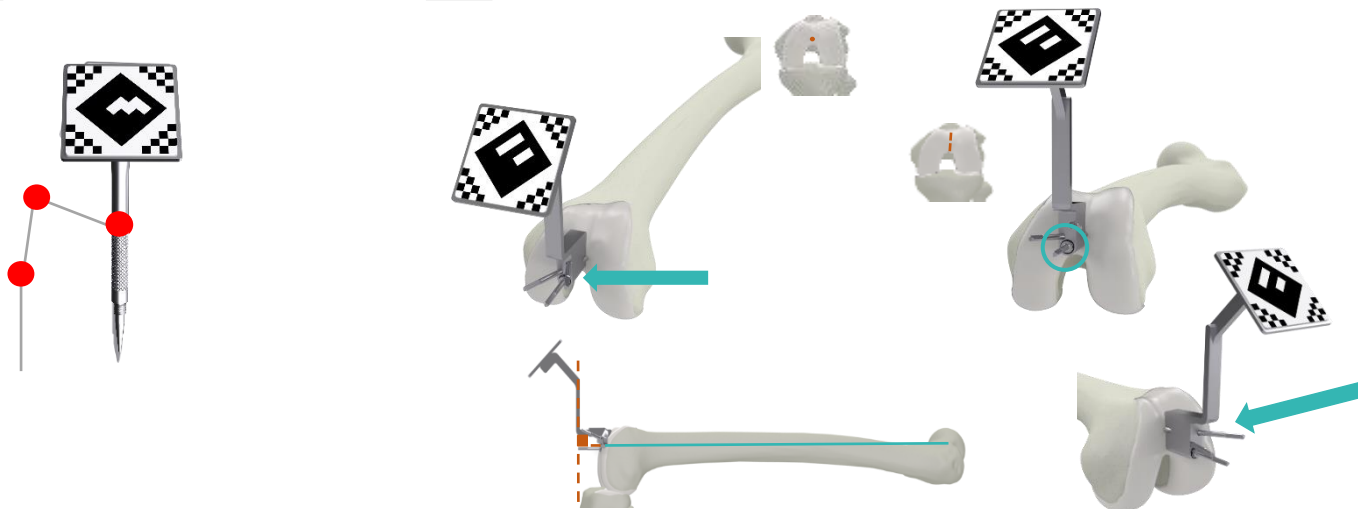


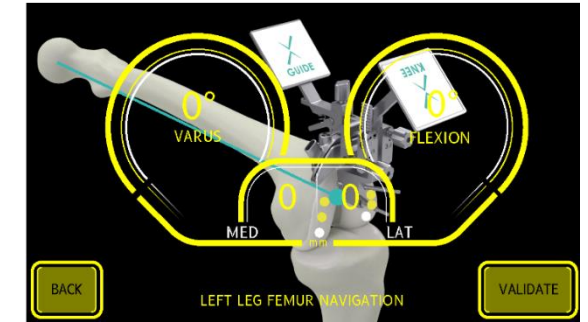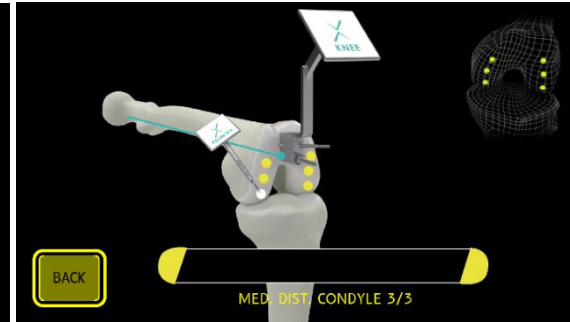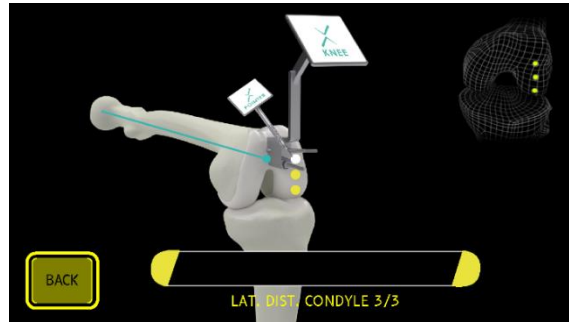
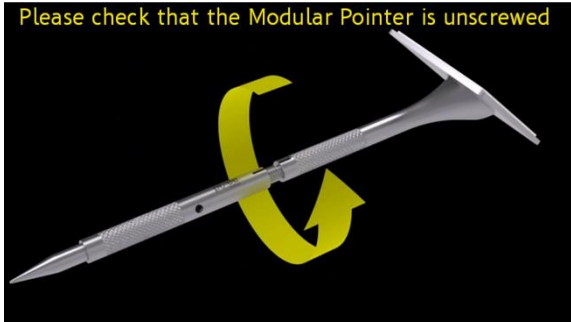


## Modular pointer des-assembly

## Lateral & Medial distal Condyles registration

## Navigation

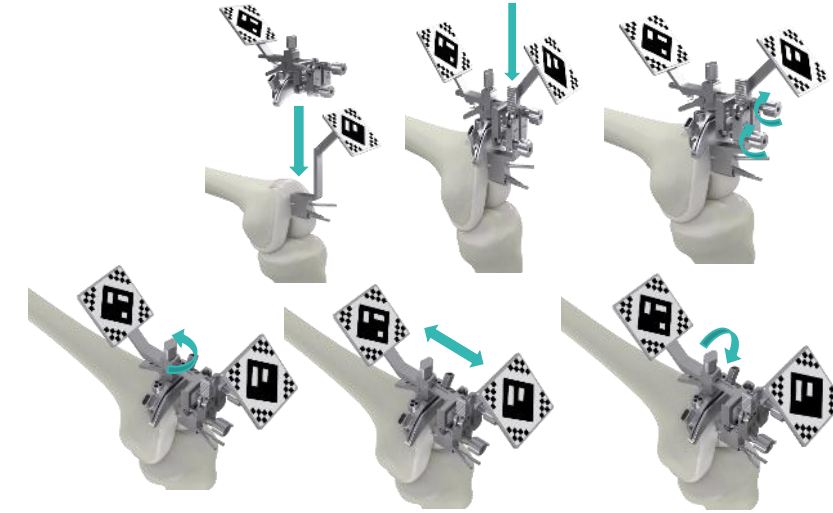

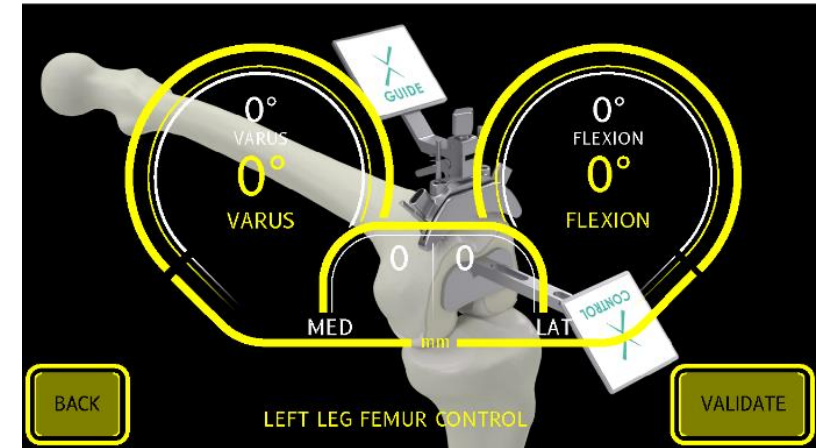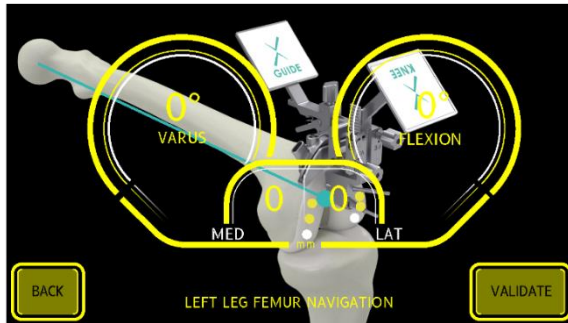

Navigation









Femoral distal Cut

Femoral distal Cut control



...

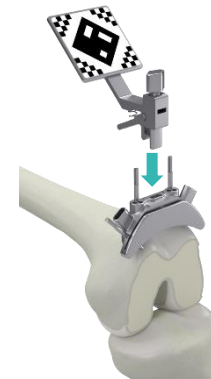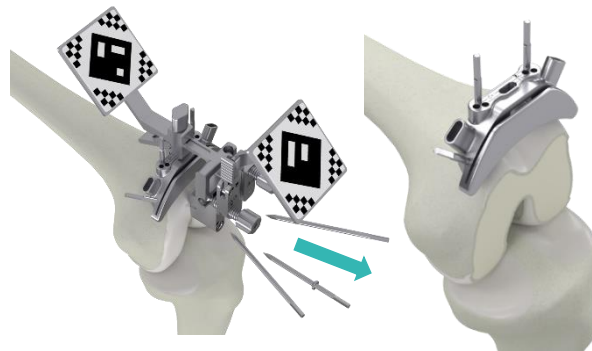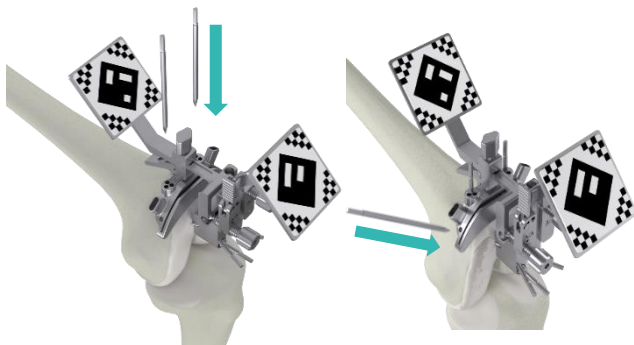

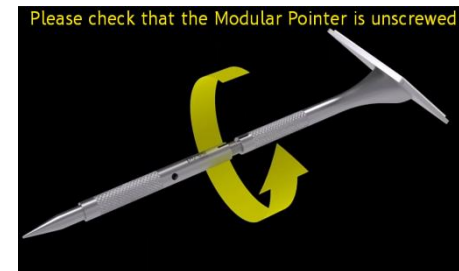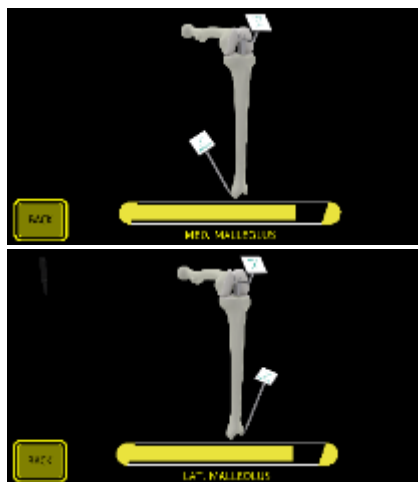
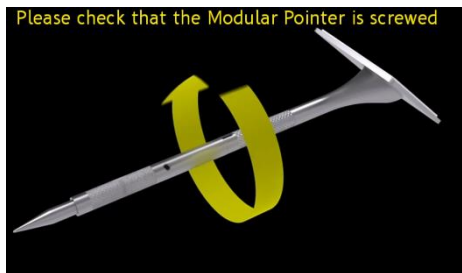





## Bone reference installation

## Modular Pointer assembly

## Malleoli registration

## Modular pointer des-assembly

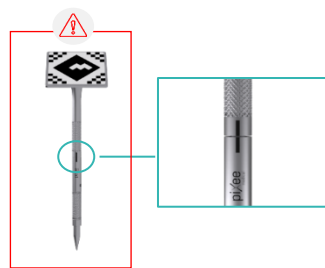
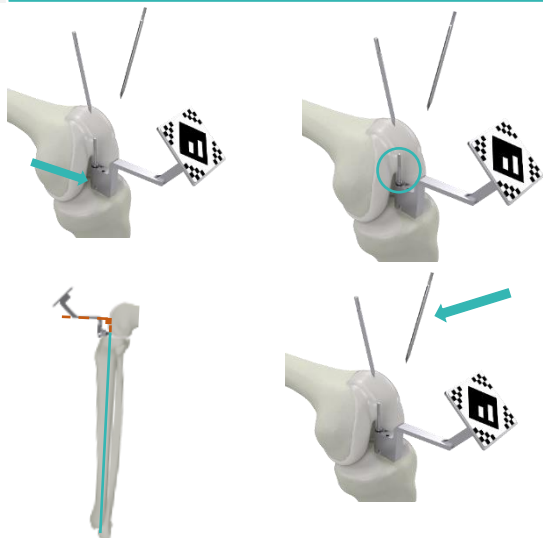


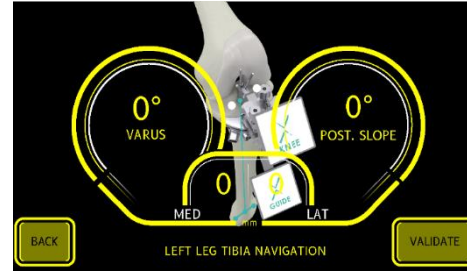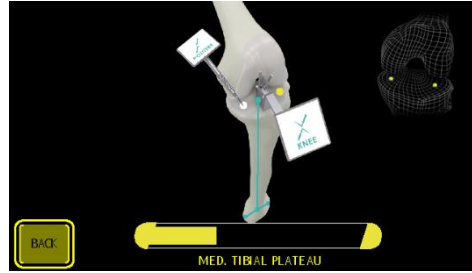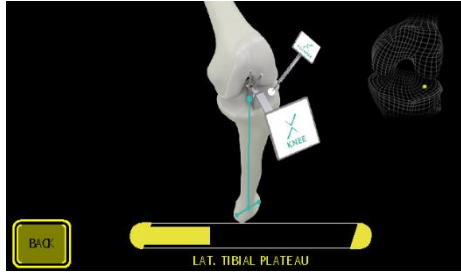



## Lateral Tibial Plateau registration

## Medial tibial plateau registration

## Proximal tibial cut navigation

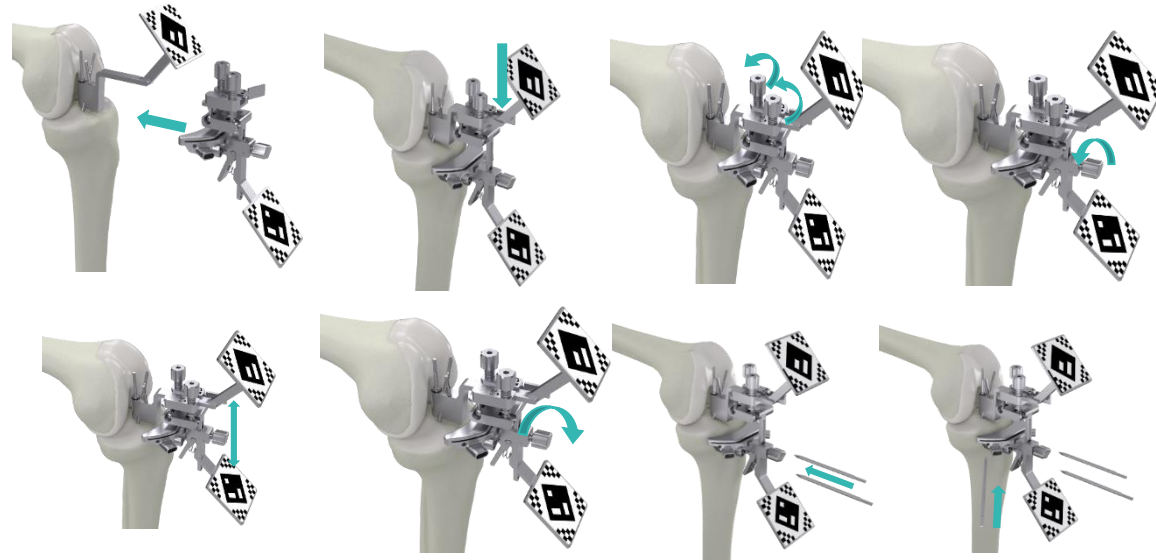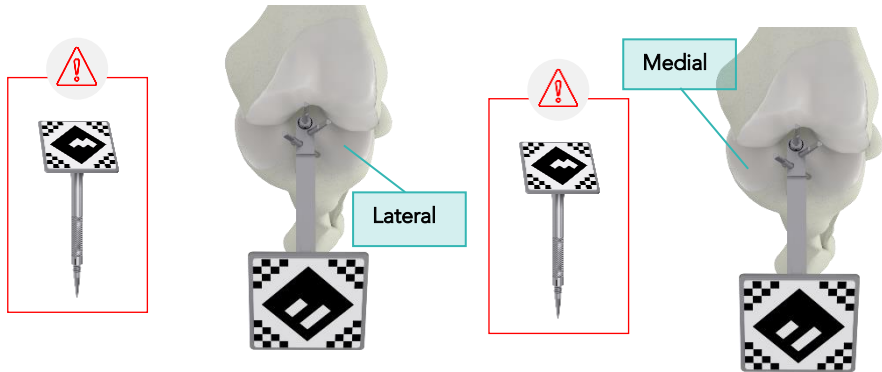



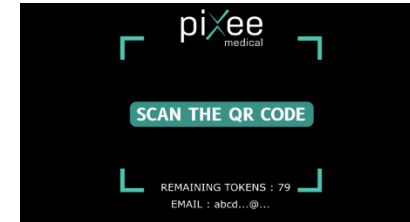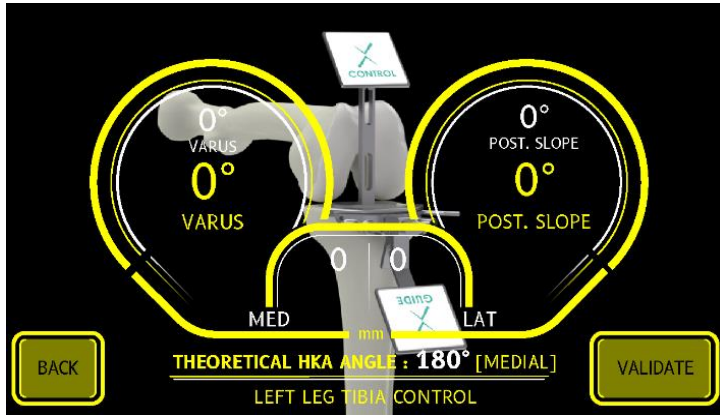


Proximal tibial cut Control 


End of protocol 

E-mail address confirmation 

Sending Status confirmation

Automatic return on starting page

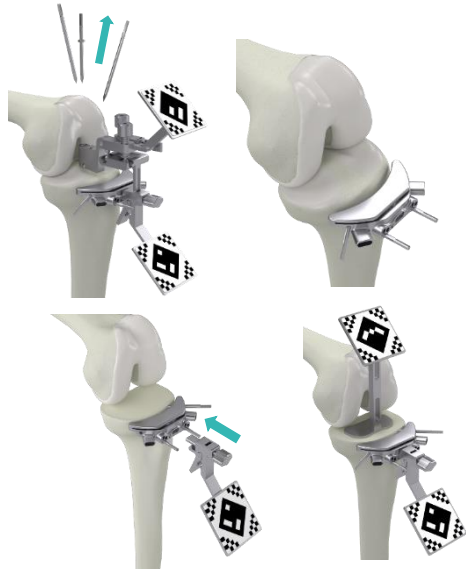

 For more information on the postpone of surgical reports, please refer to the slide : *How to send surgical reports after a surgery ?*



A new vision for computer  
assisted orthopedic surgery.



+33 (0)4 58 10 13 65



contact@pixee-medical.com



PIXEE MEDICAL  
Temis Innovation  
18, rue Alain Savary  
25000 BESANCON  
France